


Efficiency of UI CUBE

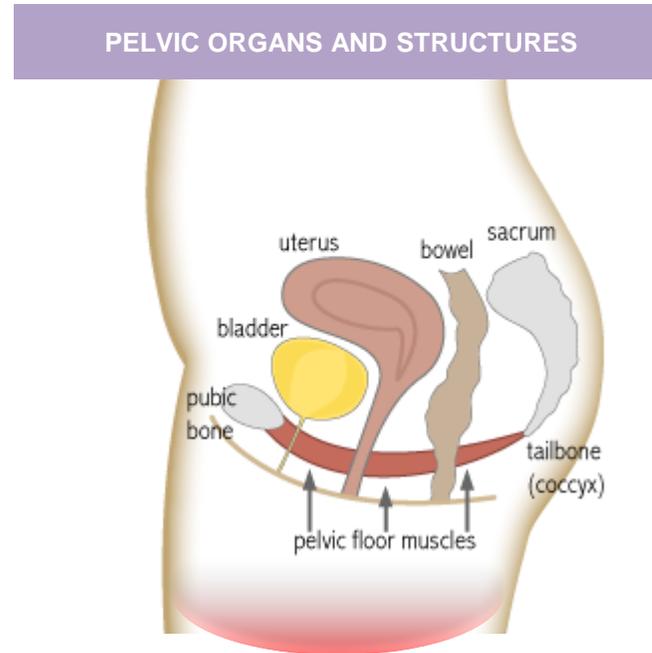

1. UI CUBE

Non-surgical treatment using magnetic field stimulation to treat urinary incontinence, improve prostate, and maximize the effect of vaginal contraction




- Electrodes are not required as current is formed as a magnetic field rather than an electrical stimulus
- There is no need for a separate electrode, so there is no need to insert a separate electrode into the vagina or anus or attach it to the skin
- No separate dressing is required, so you can proceed with comfortable treatment without discomfort.
- Since there is no energy attenuation due to resistance, it has an even effect on the deep nerve tissue

2. Existing Treatments & UI CUBE

Existing Treatments	UI CUBE
<p data-bbox="206 486 715 658">Low Frequency</p> <p data-bbox="206 725 715 896">Laser /HIFU/RF</p> <p data-bbox="206 953 715 1125">Surgical therapy</p>	<div data-bbox="1039 379 1192 525">  <p data-bbox="1039 544 1192 572">Anesthesia</p> </div> <div data-bbox="1039 608 1192 733">  <p data-bbox="1039 739 1192 768">Invasive</p> </div> <div data-bbox="1054 815 1177 961">  <p data-bbox="1026 986 1205 1015">Undressing</p> </div> <div data-bbox="1026 1068 1182 1208">  <p data-bbox="1026 1210 1205 1239">Implantable</p> </div> <div data-bbox="1319 365 1482 525">  <p data-bbox="1319 544 1482 572">Anesthesia</p> </div> <div data-bbox="1319 591 1482 751">  <p data-bbox="1319 756 1482 785">Invasive</p> </div> <div data-bbox="1319 815 1482 975">  <p data-bbox="1307 986 1485 1015">Undressing</p> </div> <div data-bbox="1319 1029 1482 1189">  <p data-bbox="1307 1210 1485 1239">Implantable</p> </div> <div data-bbox="1722 508 2155 1039">  </div> <div data-bbox="1600 1168 2351 1253">  <p data-bbox="1765 1200 2351 1239">Kegel Exercise of 12,000 times</p> </div>

3. The effect of UI CUBE

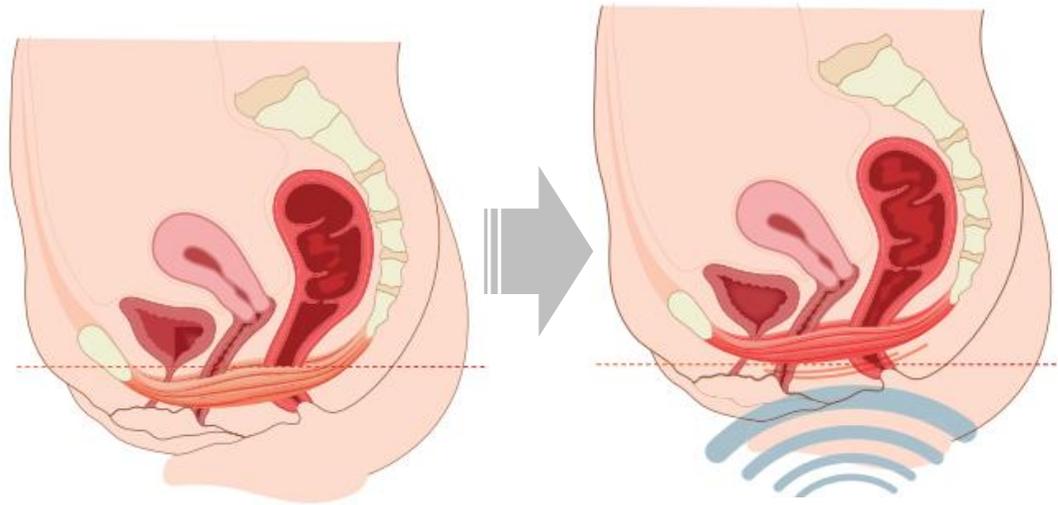

Before Treatment

After Treatment

❖ Comparison Before and After Pelvic Floor Muscle Exercise



[Before Pelvic Floor Muscle Exercise]

[After Pelvic Floor Muscle Exercise]

The treatment for 30 minutes can be the effect of 12,000 Kegel exercises and improves voluntary muscle strength by strengthening the pelvic floor muscles, the root cause of urinary.

4. How to use UI CUBE.

1. Connect the power cord to the power input terminal on the back of the device.
2. Once you push the power button, the LED is turned on as like picture below.
3. You can feel free to sit on the device without undressing as like the picture below.
4. Push the "Mode" button to choose what modes you want. .
5. Choose whether to operate the output by using the On/Off button.
6. Use the Strength button to set the strength of the magnetic field.
7. Once the treatment is finished, the Off LED of On/Off button is displayed, and it stops working.



5. UI CUBE, Q&A



1. Is there any pain in the treatment?

Because it is treated in a non-invasive process, most patients do not feel any discomfort or pain during treatment.

2. When can patients feel effects after treatment?

In some cases, patients can feel the immediate effects but 2 or 3 times of treatments is recommended for 3 to 4 weeks.

3. When can patients return to their normal activities after the treatment?

Any side effects or pain after treatment do not occur so that patients can return to work or normal life immediately after treatment.

4. Can female patients get treatment during period?

We recommend receiving advised by doctors in charge.

